

EVAC+CHAIR®
200H

CARRYLITE
TRANSIT CHAIR

USER GUIDE



EVAC+CHAIR®

THE WORLD'S NO.1 STAIRWAY EVACUATION CHAIR SINCE 1982

200H Main features

Weight Capacity



400lbs (181kg)



1 Lightweight aluminum construction

Increased strength and robustness for continued use

2 Integrated handles (no pinch points)

For effective grip when lifting and transporting the passenger

3 Large wheels

For increased stability when transporting over rough terrain

4 Unique locking system

When deployed it automatically locks chair into it's useable position

5 Adjustable patient straps


For increased passenger safety

6 Footrest

For passenger comfort during transportation

Dimensions	Height	Width	Depth	Weight
Folded	23"	9"	11"	16lbs

Opening the Evac+Chair[®] 200H

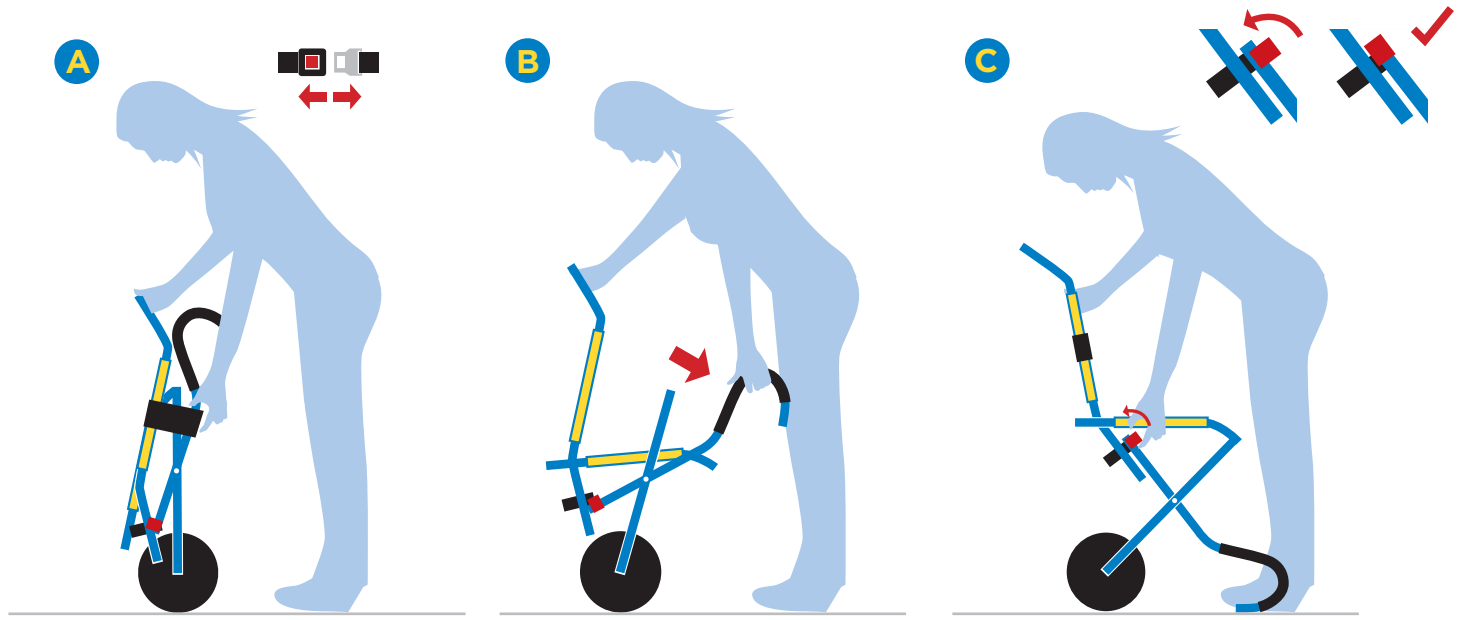


Hold chair upright with one hand and release the strap buckle.

Holding chair with one hand, use other hand to start to unfold the chair.

Hold the chair with both hands and fully deploy the chair. Rotate and apply the red safety catches to lock chair into place.

200H Passenger transfer

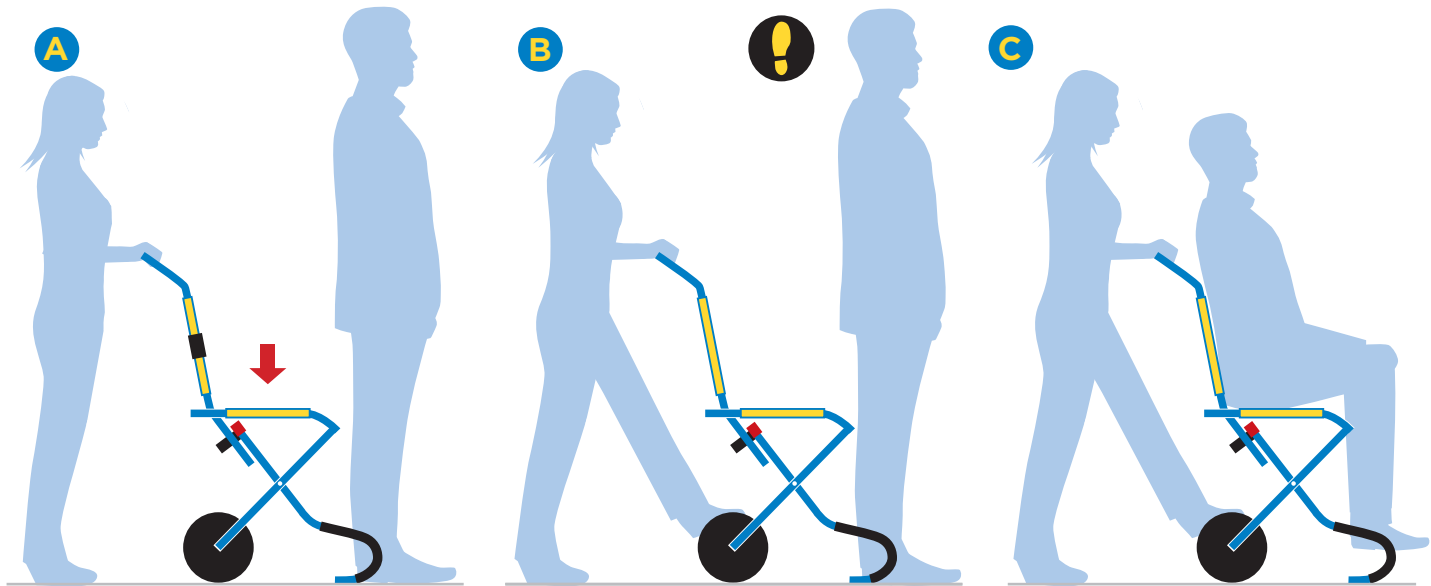


Stand behind chair and hold chair upright with both hands.

While holding chair place one foot on rear axle.

Keep chair upright for passenger to be seated or as passenger is assisted in being seated.

**Note: Occupant should never be left unattended*

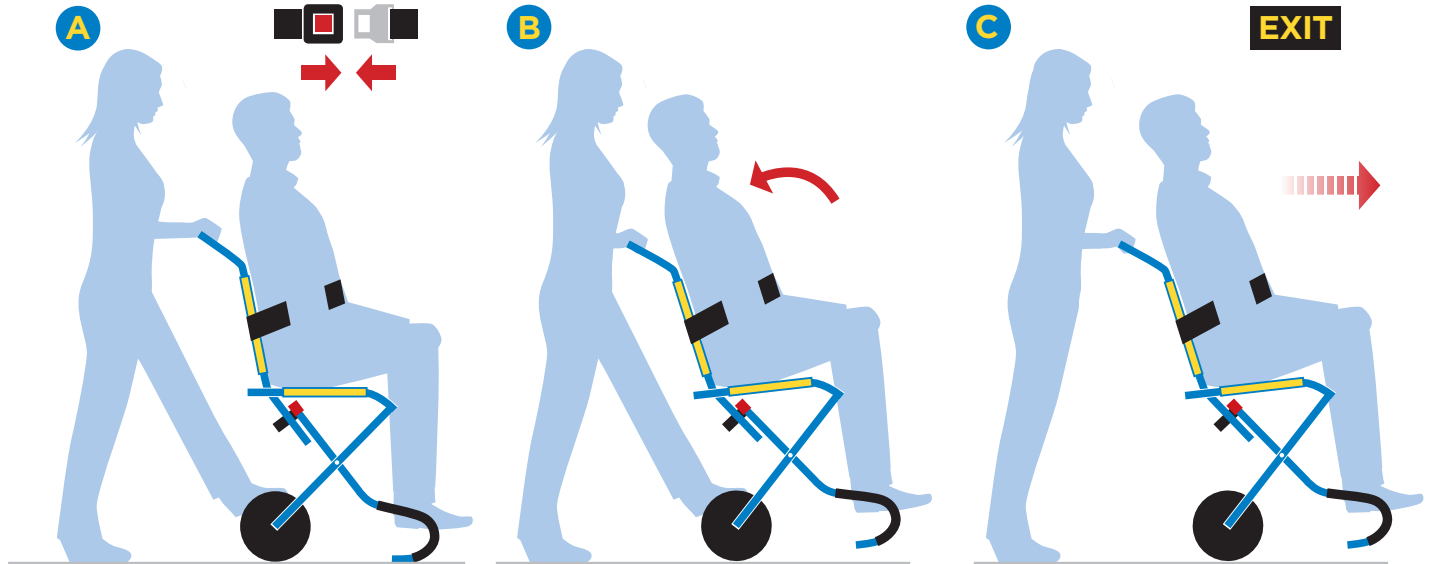


Passenger to fasten seatbelt, if necessary seek assistance.

Holding chair with both hands and with one foot on rear axle tilt chair slightly backwards and hold position.

Move towards the ultimate point of safety.

200H Closing the Evac+Chair®

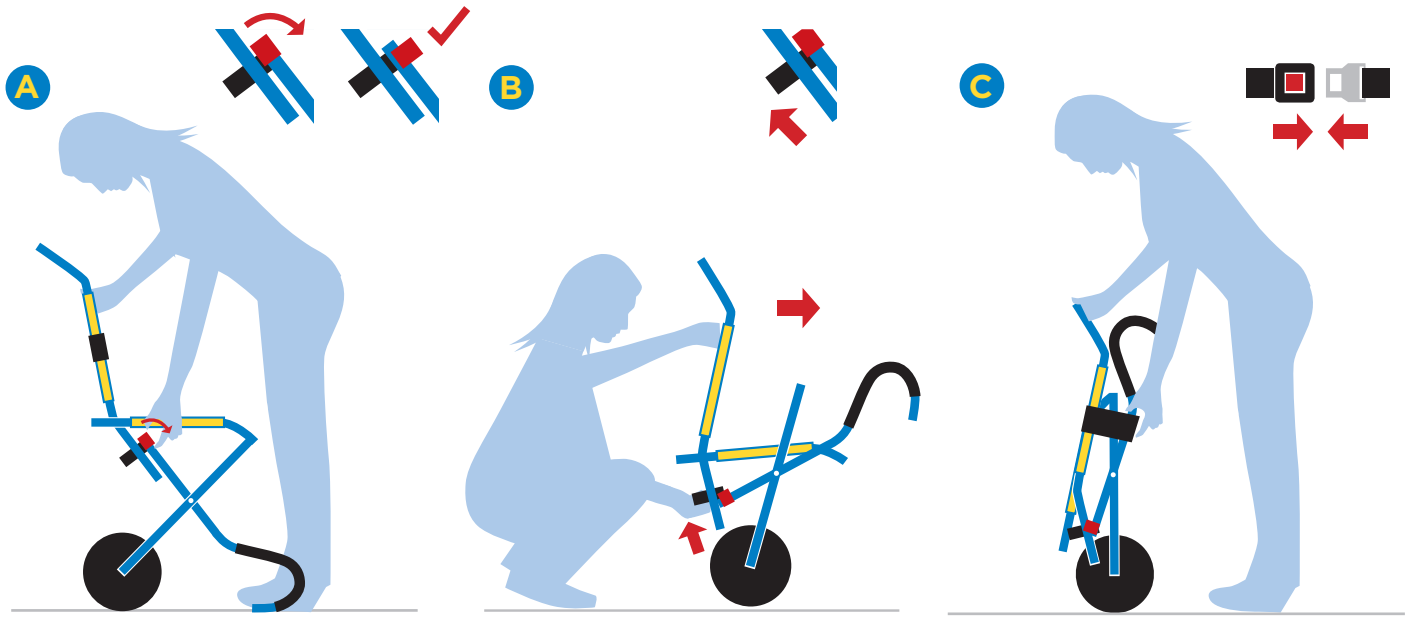


Rotate and release the red safety catches.

Hold the chair and tilt forwards and deploy the release handle to allow folding of chair.

Holding the chair with both hands complete folding. Finally wrap and fasten the strap buckle.


Installation for wall storage 200H

A

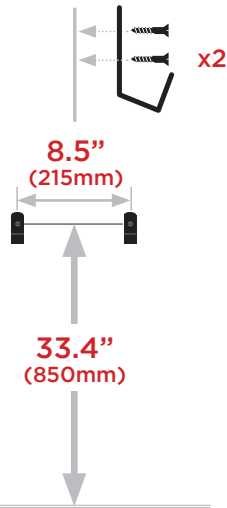


When using the wall hooks to store the chair, fix hooks to wall using 2 screws in each hook to the above dimensions.

B

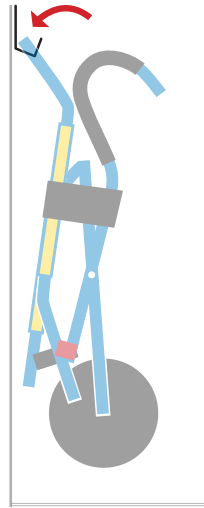

Once wall bracket has been fixed in place the chair can be installed by hanging chair by its handle.

C

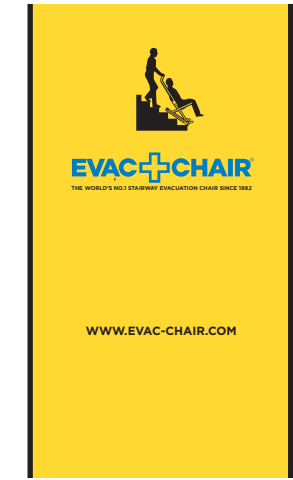

Dust cover can now be placed over the installed Evac+Chair® for added protection.

EVAC+CHAIR®

THE WORLD'S NO.1 STAIRWAY EVACUATION CHAIR SINCE 1982



Please scan QR code for online User Guide

For further information please contact:

Evac+Chair North America LLC

3000 Marcus Ave, Suite #3E6,
Lake Success, NY 11042-1012

T 516-502-4240

W www.evac-chair.com

E sales@evac-chair.com



EVAC+CHAIR® AND ITS SILHOUETTE SYMBOL ARE REGISTERED TRADEMARKS. COPYRIGHT BY EVAC+CHAIR® NORTH AMERICA LLC. ALL RIGHTS RESERVED.